dutchblue.com


BATH CHAIRS WITH BACKREST

User Manual | Bath chair with backrest
Gebrauchsanweisung | Duschhocker mit Rückenlehne
Mode d'emploi | Chaise de douche avec dossier
Gebruikershandleiding | Douchekruk met rugsteun
Ръководство за употреба | Стол за баня с облегалка

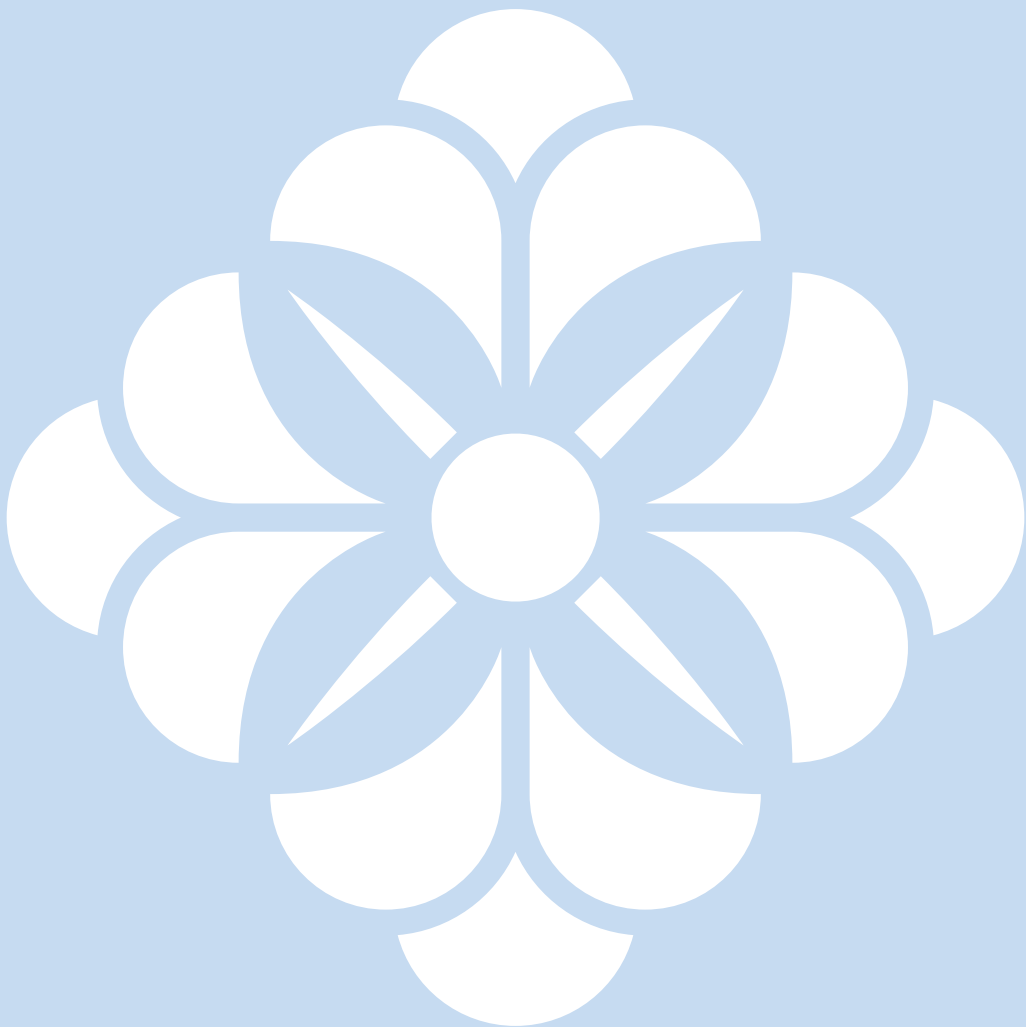REF 7507


BATH CHAIRIES WITH BACKREST

English	User Manual	5
Deutsch	Gebrauchsanweisung	7
Français	Mode d'emploi	9
Nederlands	Gebruikershandleiding	11
Български	Ръководство за употреба	13

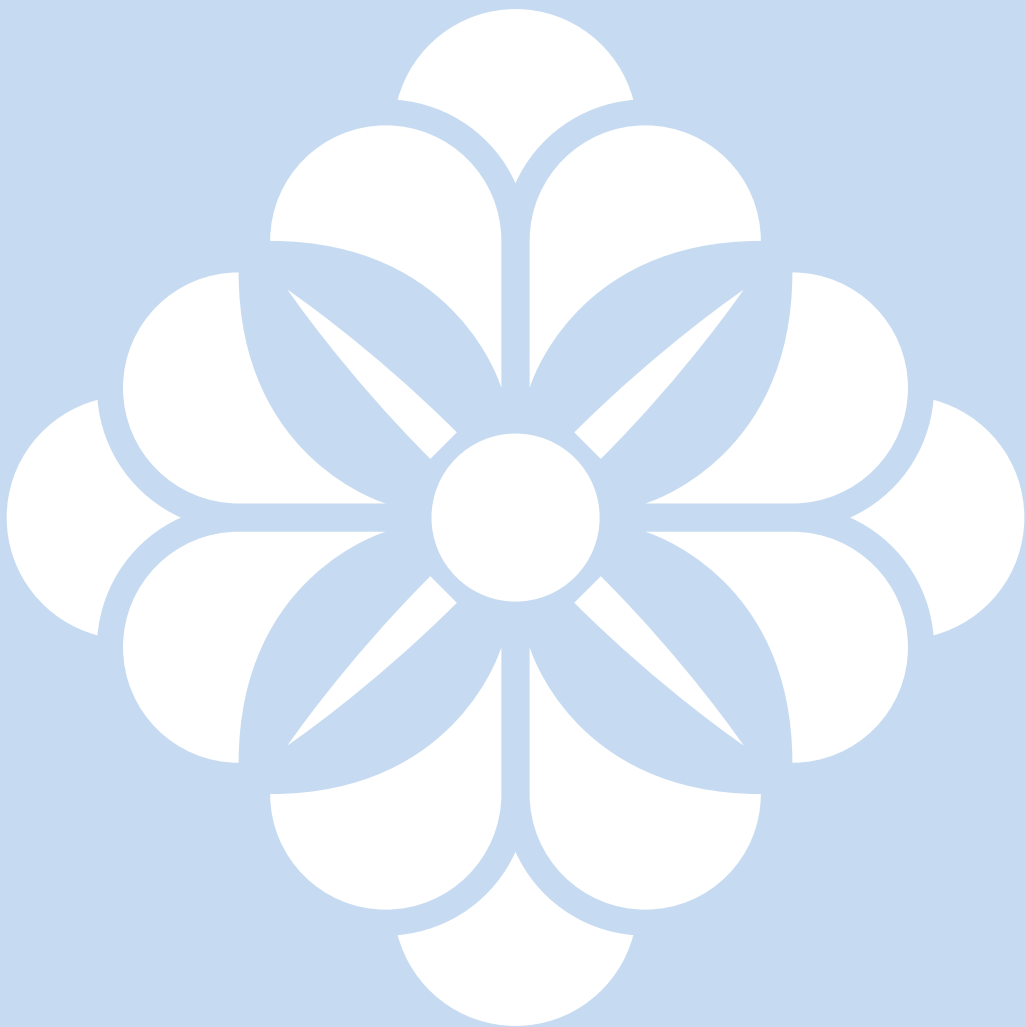ENGLISH

Bath chair with backrest

Water-permeable bath chair with backrest, anti-slip seat and non-slip knobs on the feet.

Material: Powder-coated aluminum frame. 5-way height adjustable seat. Maximum user weight 100 kg. Ideal for every shower.

Instructions for assembly

1. Remove all the parts from the box and check that they are in good condition.
2. Secure the feet to the bottom of the stool with the included screws and washers.
3. Adjust seat height to desired size.
4. Screw the U-piece to the seat base.
5. Insert the backrest into the U-piece and adjust with a safety clip.

Warnings, Indications, and Precautions

1. Make sure the chair is fully assembled after use.
2. Make sure all legs are the same height.
3. Ensure that factory-pre-installed parts are properly secured.
4. Ensure proper use on a flat, stable surface.
5. If you notice anything unusual, contact the point of sale.
6. Do not exceed the specified maximum load.

Wash and dry the product after each use. Remove dirt with a damp cloth and neutral cleaner and let dry. Do not use corrosive or abrasive chemical cleaners.

They can damage the material, reducing your strength and ultimate safety.

Dispose of items in a responsible and appropriate manner at the end of their useful life, preferably returning them to a clean place. In all cases, municipal ordinances must be observed.

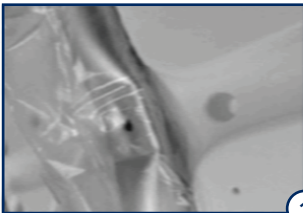
User information: Any serious incident related to the product must be reported to the competent authorities of the manufacturer and the Member State in which the user is located.


ENGLISH

1. There are two U-shaped rods (legs). The one with the crack is the one that "touches the seat." Attach the other rod to the front center screw.
2. Place the U-piece at the back and screw it to the two holes in the middle and back, securing it with long screws.
3. Attach the L-shaped bar to the back of the chair with two holding points. Use small screws.

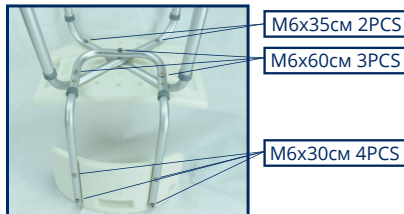4. Attach the L-shaped bar to the back of the chair with two holding points. Use small screws.

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Scheitlin-Papier AG