dutchblue.com

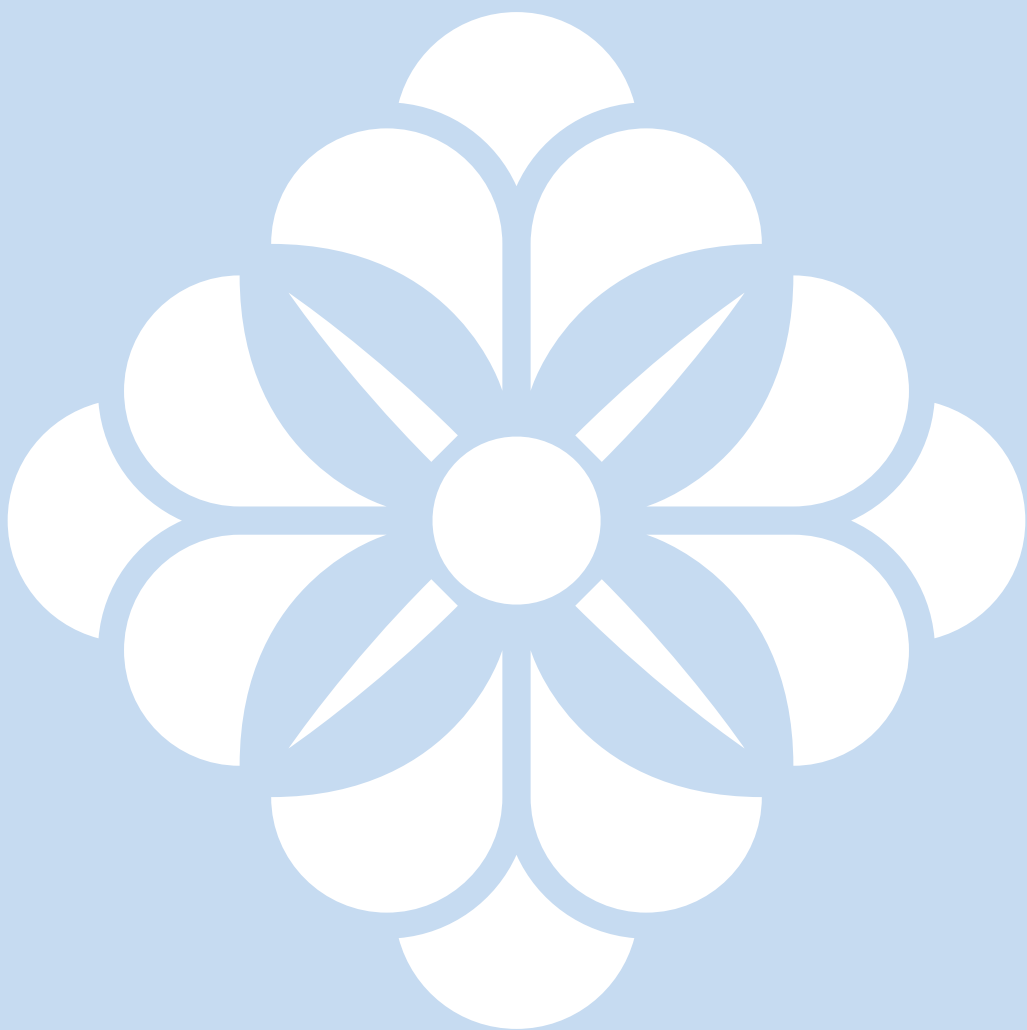
DREAMIES

User Manual | Viscoelastic mattress
Gebrauchsanweisung | Viscoelastische Matratze
Mode d'emploi | Matelas viscoélastique
Gebruikershandleiding | Visco-elastisch matras
Ръководство за употреба | Вискоеластичен матрак

REF 3020, 3030

DREAMIES

English	User Manual	4
Deutsch	Gebrauchsanweisung	5
Français	Mode d'emploi	6
Nederlands	Gebruikershandleiding	7
Български	Ръководство за употреба	8

ENGLISH

DREAMIES viscoelastic mattress

DREAMIES nursing mattresses provide a high degree of comfort. They mould themselves to the user's contours while offering optimum support of the entire body. The top layer of the mattress is made from a special type of viscoelastic memory foam that facilitates an even distribution of pressure and body weight. The top layer of PREMIUM mattresses is made from a sturdier layer of viscoelastic memory foam that facilitates an even better distribution of pressure and body weight. This improves circulation of the blood and the supply of oxygen to the tissues. The sturdy bottom layer ensures that the entire body is supported to the greatest extent possible. Effective for decubitus prevention.

Composition

Mattress

- 4 cm viscoelastic memory foam
50 kg/m³ (Basic) 85 kg/m³ (Premium)
- 8 cm polyurethane foam

Mattress cover

- | | |
|--------|----------------|
| Top | 100% PU |
| Bottom | 100% polyester |

Instructions for use

Remove the packaging.
Do not use scissors, knives or other sharp implements when opening the packaging. Unroll the mattress and leave unused for one day, in order for the memory foam to fully expand.

Instructions for cleaning

The mattress cannot be washed, but can be aired out. The cover can be cleaned with a soft cloth and mild detergent or clinical cleaning products. Never use disinfection agents, phenol or solvents.

In addition, the protective cover can be washed up to 90°C. Autoclavable, suitable for tumble dryers, do not use industrial dryers.

PLEASE NOTE:

- The typical odour of visco-elastic foam will disappear in time.
- Avoid contact with sharp objects.
- Flame retardant according to Crib 5.
- Maximum patient weight: 150 kg.
- Dimensions: 90x200x12 cm.

Longer than 6 months, the inner materials meet great pressure and challenge to rebound and lose the height.

Vacuum products will lose about 5% height after opened the package based on the special package way.

Please remove it from the package within 6 months of receipt.

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Scheitlin-Papier AG