dutchblue.com

# SHOWER CHAIRS

User Manual | Shower commode chair  
Gebrauchsanweisung | Dusch- und Toilettenstuhl  
Mode d'emploi | Chaise de douche et de toilette  
Gebruikershandleiding | Douche en toiletstoel  
Ръководство за употреба | Стол за душ и тоалетна

REF 5060

# SHOWER CHAIRIES

<b>English</b>	User Manual	3
<b>Deutsch</b>	Gebrauchsanweisung	11
<b>Français</b>	Mode d'emploi	19
<b>Nederlands</b>	Gebruikershandleiding	27
<b>Български</b>	Ръководство за употреба	35

**EN** Read this User Manual before using the device for the first time and keep it at hand for the user as well as for future use and subsequent sales.

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# ENGLISH

## Shower commode chair

Easier shower and toilet care, fits over most standard size toilets. Designed for an easy and fast hygienic routine. PU-foam seat with hygiene cutout. Easy side entry and exit due to swing-away armrests. Height-adjustable and swing-away footrests. High-quality caster wheels with brakes.

The Shower commode chair is a medical device indicated for use by persons with limited mobility who are unable to stand, walk and /or seat independently. The device is specifically designed for individuals who require additional support and stability during washing, toileting and other hygienic activities. It is a class 1 medical device, which is intended to be used only as seating option for toilet issues and showering. Its purpose is to help physically handicapped people during their hygienic activities. Any other use is prohibited.

## Parts and description

- Frame Material: Aluminum
- PU seat panel, backrest and armrest pad.
- Fits over most standard toilets.
- 5" TPR wheels with brakes
- Including toilet bucket with lid.
- Product size: W57/D82/H 92.5 - 97.5 cm
- Seat size: W47/D42.5/H50 - 55 cm
- Tube diameter: 22.2 mm
- Tube thickness: 1.25 mm
- Net weight: 11.35 kg
- Safety loading Capacity: 125 kg

## ENGLISH

Components: Seat Panel - 1 pc; Backrest - 1pc; Main Frame - 1 pc; Armrest - 2 pcs;  
Toilet bucket - 1 pc.

Accessories: Caster - 4 pcs; Footrest - 2 pcs; Screw A - 4 pcs; Screw Nut B - 4 pcs;  
Black Knob - 2 pcs; Waterproof screw cap - 4 pcs; Gasket - 4 pcs; Fixing Nut - 4 pcs;  
Hexagonal screwdriver - 1 pc; Spanner - 1 pc; C-clamp - 2 pcs.

### Instructions for assembly

1. Open the main frame and put it straight.
2. Cover the backrest on both sides of the pipe mouth, and then tighten both sides with the black knobs.
3. Install the backrest as shown in the picture.

4. Cover the armrest to one side and press it to fix tight. And then insert the screw A to the holes on both up and down plastic parts.
5. Put on the screw nut B inside of the screw and tighten it with hexagonal screwdriver and the spanner.
6. Put on the black waterproof cap on the screw nut after installation.

## ENGLISH

7. This picture shows how the armrests are fixed.

8. Reverse the main frame to install the wheels.

9. Put on the brakes of the wheels, and put on the fixing washer, put on the gasket, tighten the wheels.

10. Insert the C- clamp to both sides of the front wheels, tighten it to finish.

11. Put the footrest on both sides of the front wheels and tighten it to the front pipe.

12. Press to lock the footrest.

## ENGLISH

13. Fix the seat plate on the back frame level bar.
14. Installation is finished.

### Safety instructions

**Please observe the following safety instructions:**

1. Before using the device for the first time, the User Manual must be read completely in order to avoid hazards or damage caused by maloperation.
2. Before each use, check the device for visible damage or deficiencies.
3. In case of malfunctions or damage of any kind, the device must not be used.

**⚠ Caution! Danger of falling!**

4. The chair must only be used with the armrests locked in place when sitting and moving around.
5. Sitting down, getting up, transferring of a patient should be done under supervision or with the help of a trained caregiver, depending on the patient's state of health.
6. Patients who are prone to uncontrolled actions or movements must be supervised when using the chair.
7. The chair must only be moved on a slope or incline with the help of a caregiver. Generally, on an uphill slope the chair should be pushed forward and on a downhill slope it should be pulled backward.

**⚠ Caution! Danger of breakage!**

8. The product is only suitable for the intended use. Observe the permitted user weight.
9. Never carry or lift the product while a patient is sitting on it.
10. Do not pull or hold on to additionally attached components or accessories.

**⚠ Caution! Danger of tipping over!**

11. The chair must only be used with the armrests locked in place when sitting and moving around.

## ENGLISH

12. The chair must only be pushed by the push handle (backrest).
13. Only move the chair at a moderate walking pace (with and without patients).
14. Do not lean out of the chair (both in standstill and while moving). If the swivel castors are in an unfavourable position (pointing inwards), stability of the chair is limited.

**⚠ Caution! Danger of pinching or catching!**

15. When attaching the seat plate, do not reach between the seat surface and the seat frame so that no body parts are trapped.
16. When pulling out and pushing in the toilet bucket, there is a risk of trapping soft tissue of the patient. To prevent injuries, the bucket may only be pulled out or pushed in when no user is sitting in the chair.

**⚠ Caution! Risk of injury!**

17. There is a risk of injury with surface temperatures > 40°C. Therefore, do not expose the product to high temperatures (e.g. sunlight, radiators, hot water). Allow the product to cool down before use.

***Note!** If the product is used by patients with insensitive skin (no temperature perception or skin damage), a heat check (e.g. touching with the back of the hand) must be carried out by the nursing staff.*

### Operation of the product

Before use, it must be ensured that the product is in proper condition. If defects or malfunctions are evident, the product must not be used. Have the defects corrected by a specialist dealer before use.

***Note!** If necessary, depending on the patient's limitations (adequate mobility and muscle strength), sitting down and getting up must be done with the help of a caregiver. Subsequent adjustments may only be made by qualified personnel.*

#### 1. Sitting down

- 1.1. Before sitting down and standing up, make sure that the product is securely positioned to prevent it from slipping or tipping over, that at least two wheels are braked, that the armrests are properly engaged, that the seat surfaces are firmly in place, and that the footrests are folded to the side or up.
- 1.2. The user must position themselves as close as possible to the product. If necessary, they can orientate or support themselves with their hands on the armrests.
- 1.3. If this is not possible due to their physical limitations, the user must seek the assistance of a competent person. This can be done with the help of an assistive device.

# ENGLISH

## **Attention! Danger of falling!**

- 1.4. Falling down or slumping on the chair seat must be avoided. The patient should sit on the full chair surface, if possible, and not only on the edge of the seat.
- 1.5. When transporting people, the entire surface of the backrest and the footrests must be used if possible.

## 2. Getting up

- 2.1. Before standing up, make sure that the product is securely positioned to prevent it from slipping or tipping over, that at least two wheels are braked, that the armrests are properly engaged, that the seat surfaces are firmly in place, and that the footrests are folded to the side or up.
- 2.2. The patient must move forward on the chair so that their feet are on the floor.
- 2.3. The patient must support themselves with their arms on the armrests and slowly rise from the seat.
- 2.4. Do not lean your body too far forward to avoid a potential risk of falling.
- 2.5. The patient should be helped to stand up by the nursing staff.

## **Caution! Danger of falling!**

- 2.6. The patient must never leave their feet on the footrests when sitting down or getting up.

## 3. Parking brake operation

- 3.1. With your foot, press down the parking brake lever. The wheel is now braked and fixed in its direction. The wheel is released by lifting the actuating lever with the foot.

## 4. Removing the footrests

- 4.1. Raise the footrest by approx. 100 mm and turn outwards by 90 degrees.
- 4.2. Now you can push the footrest all the way up and pull it out.

## 5. Handling of the toilet bucket

- 5.1. From the front of the chair, pull the toilet bucket under the seat. To remove it, lift the bucket slightly and then pull it forward.
- 5.2. You can alternatively insert or remove the toilet bucket from above by lifting the seat.

# ENGLISH

## 6. Removing the shower / toilet seat

- 6.1. The seat can be removed from its locked position on the chair frame by lifting it off for cleaning or replacement.

## Care and maintenance

### 1. Cleaning and care

- 1.1. The product can be cleaned using a mild conventional cleaning and disinfecting agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.
- 1.2. Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.

*Note! Keep the product clean and clean it regularly. Special attention must be paid to hygiene in the sensitive areas.*

- 1.3. The product is suitable for manual cleaning and disinfection.

### 2. Maintenance

- 2.1. Wear and tear that is not recognised in time and/or improper handling, as well as irregular or no maintenance may result in accidents.

*Note! For safety reasons, the product must be professionally maintained at least once a year.*

- 2.2. Check product for visible damage (cracks, breakage, loose parts).
- 2.3. Check stability before use.
- 2.4. Maintenance should include the following inspection work and should be documented.
- 2.5. All bolted connections for tightness (especially the wheel inserts and the frame).
- 2.6. Function of the armrests and locking bolts.
- 2.7. Function of the footrest (replace defective footrests).
- 2.8. Running characteristics and tread of the wheels (replace defective wheels).
- 2.9. Function of the parking brakes (replace defective brakes).
- 2.10. Inspect accessories for deficiencies.

## Symbols & nameplates

The nameplate must not be removed! For ordering spare parts and processing complaints, the device name, serial number and year of manufacture will be required.

Read the User Manual!

Do not dispose of  
as household waste

Caution! Observe the  
safety instructions

# ENGLISH

**SHOWER CHAIRIES**  
dutchblue.com

**REF 5060**  
1 (1 x 1)

Dutchblue.com LTD  
Parizhka Komunda 26,  
Varna, Bulgaria

   **DoM:** 07.2022  
**SWL:** 125kgs

290620225060 3 800234 873794

The package must be kept in a dry environment

Product complies with (EU) 2017/745 MDR

Medical device label. This product is a medical device

Manufacturer

**REF 5060**

Article number

**DoM:** 07.2022

Year of manufacture

**SWL:** 125kgs

Safety Working Load-  
Maximum user weight in KG

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