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Technical Data Sheet

MaiMed® – FFP2 Atemschutzmaske

Produktbeschreibung Product Description

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The filtering half mask is intended for protection against solid and oily particulates (such as coal dust, cement dust) and liquids. Single-use..

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Check prior to use: The mask body is in good condition and without deformation. The ear straps are attached well on the mask. The nose clip is attached well on the mask. The product is within expiry date. See information supplied by the manufacturer.

How to wear the mask: Unfold the mask. Hold the ear straps with both hands and position the mask under the chin, covering mouth and nose. Put the ear straps on ears. Using both hands, mould the metal nose clip to the shape of nose. To check for proper fit, cup both hands over the mask and exhale vigorously. If air leaks around the nose, tighten the nose clip; if air leaks around the edge, reposition the ear straps for better fit. Double check the seal and repeat the procedure until the mask is sealed properly.

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Artikelbeschreibung/ Article Description	Größe/ Size	Farbe/ Colour	Inhalt/ Content	Artikelnummer/ Article Number	GTIN	PZN	VE/ SU
FFP2 Halbmaske/ FFP2 half mask	15,5 cm x 10,5 cm	weiß/ white	10 Stk./ pcs.	75336	4046153142507	-	80

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FFP2 half mask according to Regulation (EU) 2016/425 for personal protective equipment, category III.

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Fifth / inner layer: Polypropylen Spunbond Nonwoven

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Nose clip: PP-coated aluminium wire, blue

Frei von: Nickel, Polyvinylchlorid, Glasfaser, Latex

Free from: Nickel, polyvinyl chloride, glass fibers, latex

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Technical Data Sheet

MaiMed® – FFP2 Atemschutzmaske

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3 Jahre
3 Years

Symbole
Symbols